

Yellofins[®] Instruction Sheet



1 Your Yellofins stirrups mount to the table rails using the rail clamps supplied.



4 The toe, knee, and opposing shoulder should be maintained in a relatively straight line during positioning. Notice that for a typical patient, the thigh and foot are abducted at approximately the same angle.



2 Locate clamps adjacent to the patient's hip joint. The boots are marked (on the bottom) for the patient's left and right.



5 **For Squeeze Handle:** Reposition the stirrups by squeezing the upper handle and release handle together. When you have reached the desired degree of lithotomy and/or abduction simply release the handle.



3 Obtain adequate help to place both legs into the stirrups simultaneously. To adjust for patient leg length, loosen boot lock handle and gently slide boot up or down the bar, slightly flexing the knees. When satisfied with the position, re-tighten the bootlock handle. Secure the bootstraps.

Important Note:

As with any unfamiliar patient positioning device you should always practice its use on a nurse, physician or other appropriate volunteer prior to using it clinically. Whenever practical, a patient should be placed in the stirrups prior to anesthesia. An alert patient can communicate normal range of motion limitations and, to some extent, such patient's opposing muscles will guard against abnormal joint movement and stretching.



Warning: Do not clean stirrup by immersing it in liquid or by using a high pressure spray. Damage to stirrup may result.

US Customer Service
(800) 433-5774
Fax (978) 263-8846
www.AllenMedical.com