


ALLEN® TABLE-POWERED BEACH CHAIR™

Set-Up Reference Sheet

1 PREPARE OR TABLE

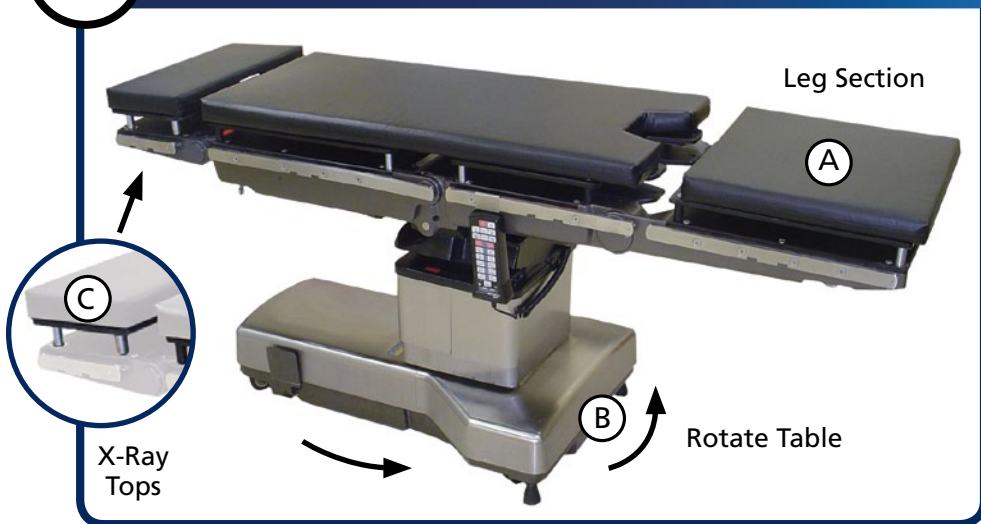

- A. Identify leg section of the table.
- B. Turn table so that leg section faces anesthesia.
- C. Remove X-ray tops.

2 ATTACH BEACH CHAIR



- A. Lower leg section all the way.
- B. Attach clamps, and ensure Finger Gauge is aligned with OR table rail.
- C. Tighten lower knob.
- D. Insert chair into clamps, and secure upper knob.

3 POSITION THE PATIENT

Following these steps will ensure the patient is in a comfortable seated position while reducing the chance of the patient sliding down the table, resulting in poor posture and spinal alignments.



- 1 Transfer the patient using standard hospital practice. Place patient's hips where the beach chair pad meets with the surgical table pad.
- 2 Place the patient in approximately 10° reverse trendelenburg.



- 3 Raise the Leg Section of the surgical table to adjust the angle of the beach chair to approximately 60°.



- 4 Flex table until there is approximately 20° of flexion in the patient's knees.
- 5 Raise the leg section for the second time to position the patient in the "Beach Chair" (modified Fowler) position.



- 6 Remove the shoulder panel on the operative side by turning the locking latch in the direction of the arrow on the back of the chair and folding the panel away from the shoulder.

THE ACCESSORIES



Universal Head Positioner
#A-90022



Intraoperative Arm Positioner
#A-92000



Counter-Traction Strap
#A-90015



LPS Arm Support
#A-90002



Easy Lock Socket
#A-40022
(2 Required)

Please see "Instructions For Use" documents for individual accessories.

For additional info, please see "Allen Beach Chair Instructions For Use" document.